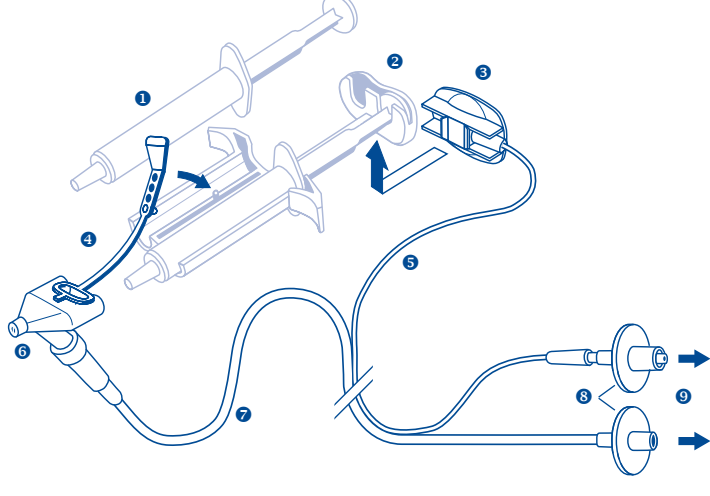
# TISSEEL/ARTISS Spray Set

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ENG

## TISSEEL/ARTISS Spray Set

For the Simultaneous Application of TISSEEL or ARTISS Two-Component Fibrin Sealant. The TISSEEL/ARTISS Spray Set is used in conjunction with a double syringe system and the EASYSpray Pressure Regulator Device.

**1. FIELD OF APPLICATION**

The double syringe systems and TISSEEL/ARTISS Spray Set have been designed for the application of TISSEEL or ARTISS fibrin sealant to broad wound areas. It has been found especially useful for indications where the application of a uniform fibrin sealant film is essential.

**2. DESCRIPTION**

Spray set consisting of:  
1 Spray head  
1 dual-lumen connection tube consisting of a pressure line with a transparent sterile filter, and a thinner sensing line with a blue sterile filter and a clip to be attached to the double syringe system plunger for gas activation.

**2.1 Principle**

Inside the spray head, which is attached to the double syringe system containing the TISSEEL or ARTISS Sealer Protein Solution and Thrombin Solution, the two solutions are passed through two adjacent outlets. These two openings are encircled by the pressure gas exit. The gas flow atomizes and mixes the two solutions. The pressure and flow rate of the pressurized gas are controlled by the EASYSpray Pressure Regulator Device. The spray set must be used only in connection with the EASYSpray device.

The gas flow is activated by occluding the opening on the clip attached to the double syringe system plunger with the thumb.

**2.2 Symbols and explanations**

- Attention, see instructions for use
- Single use
- Do not use if the product, its sterile barrier system or its packaging is damaged or shows any sign of deterioration
- Latex free
- Sterilized by ethylene oxide
- Temperature limitation range (2 – 30°C, 36 – 86°F)
- Manufacturer
- Batch number
- Use by
- REF** Catalogue number

Non-pyrogenic in unopened and undamaged package.

**3. OPERATING INSTRUCTIONS**

**3.1** Prepare the double syringe system containing the TISSEEL or ARTISS Sealer Protein Solution and the Thrombin Solution.

**NOTE:** Remove surplus air from the syringes.

**3.2** Attach the spray head firmly to the cones of the double syringe system. Fasten the pull strap to the double syringe system. The spray head is secured tightly. Should the pull strap tear, further use is still possible after careful examination of the connection site. In such a case, however, there is a potential risk of fluid leakage.

**3.3** Fit the pressure line to the lower luer-lock connector of the spray head. Connect the transparent sterile filter at the other end of the tube to the appropriate connector on the EASYSpray device. Connect the EasySpray device to the gas source according to the corresponding IfU

**3.4** Attach the clip of the sensor line by sliding it into the grooves on the back end of the double syringe system plunger. The clip is locked onto the plunger when an audible "click" is heard.

**3.5** Connect the blue sterile filter at the other end of the line to the appropriate luer-lock connector (blue ring) on the EASYSpray device.

**3.6** To activate the flow of gas, occlude the opening in the clip center with thumb. A pressure of 1.5 – 2 bar (21.8 – 29 psi) has proved suitable. With that 5 – 20 litres of sterile gas are released per minute.

**3.7** The stream of sterile gas ejected from the spray head can be used to displace the tissue fluid on the operation area, if necessary, and thus create optimum conditions for the application of TISSEEL or ARTISS Fibrin Sealant.

**3.8** The quantity of TISSEEL or ARTISS Fibrin Sealant is controlled by depressing the double syringe system plunger.

**3.9** The EASYSpray device will continue to emit gas for a brief period after the thumb is removed from the clip/plunger. This delay helps to avoid clogging of the spray head.

**3.10** To achieve an optimal result, a spraying distance of 10 – 15 cm is recommended. In specific situations where spraying at less than 10 cm is necessary, the gas pressure must be reduced appropriately.

**3.11** TISSEEL or ARTISS Fibrin Sealant, containing 1 ml of Sealer Protein solution and 1 ml of Thrombin Solution is sufficient for spraying an area of up to 100 cm<sup>2</sup>, depending on the desired film thickness.

**4. WARNING**

Any application of pressurized gas may be associated with a potential risk of air embolism, tissue embolism or gas entrapment with compression, which may be life-threatening. Especially in case of low central venous pressure there is an increased risk of air embolism in vena cava anastomoses. Be sure to take appropriate measures to exclude these risks, by control of maximum pressure (2 bar / 29 psi) and minimum distance (10 cm).

The spray set must only be used in connection with the EASYSpray device. DO NOT connect the spray set directly to the pressurized gas supply. The use of accessories of other manufacturers is not permitted.

**5. ACCESSORIES**

EASYSpray Pressure Regulator

BAXTER, TISSEEL, ARTISS and EASYSpray are trademarks of Baxter International Inc., its subsidiaries or affiliates.

Baxter AG, 1221 Vienna, Austria

**USA** Distributed in the USA by:  
Baxter Healthcare Corporation  
Westlake Village, CA 91362 USA  
1-800-423-2090

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© 2017 Baxter International Inc. All rights reserved. EASYSPRAY, TISSEEL and ARTISS are trademarks of Baxter International Inc.
Baxter AG, 1221 Viyana, Avusturya